

JUNIOR APPLICATOR - For All Combi 2000[®] Ear Tags

Please follow this guide to ensure the tag is fitted correctly Note: when applying electronic tags, remove the black piece from the bottom jaw



Locate the female part under the correct clip. The back of the female tag slots into the applicator.



Locate the male part of the tag into the applicator pin and seat correctly.



Line up the male and female parts of the tag and squeeze the applicator to clip together.



Once the tag is applied, release the handle, to allow the male part of the applicator to retract.

Important - When fixing tags to young stock, room must be allowed for growth of ears
If you require the RPA Eartag Feedback Form, please contact your supplier direct